




# INSTRUCTIONS FOR USE OF THE BONE MARROW HARVESTING SET BEST BTT®

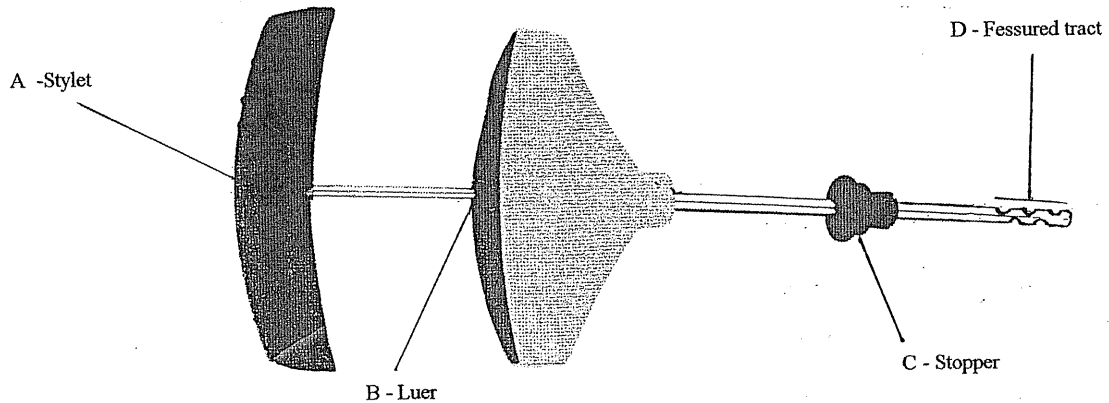

- WARNING:**
- Store in a dry clean place away from heat sources
  - Before use ensure that the packaging is intact: if perforated, damaged or open discard the instrument
  - Check the expiry date and that the model corresponds
  - Peel open the package
  - Pay attention to not compromise the sterility of the instrument before use
  - In the event of a product defect or malfunction complete the yellow claim form to be found in the product carton and conserve the defective instrument

**N.B.:**

The use of this instrument is reserved to qualified medical personnel who are aware of its application.  
The instrument is sterile (sterilized by ETO GAS) single use and should be discarded after use (Circ.Min:Sanità 403/9.5/1146-L. 10.81)  
**Sterylab declines any and all responsibility in the event of improper use or reutilisation.**

## BIOPTIC PROCEDURE

When removing the plastic protective sleeve pay attention to not remove also the stopper "B"

Some doctors flush the inside of the instrument with an anti-coagulant (heparin) before performing the harvesting procedure

- 1) Penetrate into the cortex up to the desired depth, remove stylet, attach a syringe (luer cone) and aspirate marrow
- 2) When modifying the depth use the sliding stopper to indicate the depth variation
- 3) If, after repeated punctures, penetration becomes difficult replace the stylet with the new one supplied which each set