**STERYLAB PRODUCT RANGE**

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For further information or a quotation please contact : Colin Walters

**SeedOS** Ltd

Radiation Oncology, Urology and General Medical Equipment and Services

26, The Maltings, Leighton Buzzard, Bedfordshire LU7 4BS, United Kingdom

Tel: +44 1525 850670 Fax: +44 1525 850685 Email: cwalters@seedos.com

Website: [www.seedos.com](http://www.seedos.com) Mobile: +44 07701059163

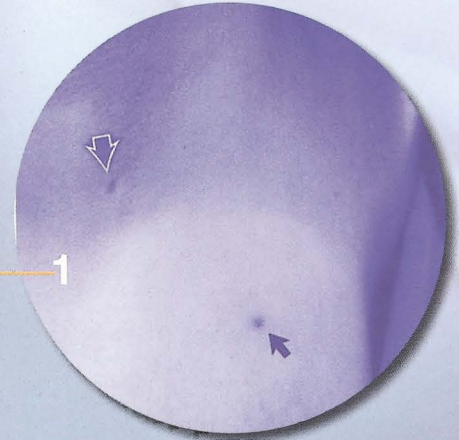
**MARKER LIQUIDO PER LOCALIZZAZIONE SENOLOGICA**  
**LIQUID MARKER FOR PREOPERATIVE BREAST LOCALIZATION**

# CARBO-REP®

Typical appearance of the skin mark (closed arrow) 7 weeks after stereotactic biopsy and carbon marking in a 52-year-old patient before surgery.

Note the similarity in size and appearance to the freckle (open arrow) in the axilla.

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Surgical specimen of a small infiltrating breast carcinoma excised from the patient. Note the infiltrating carcinoma (thick arrow) with adjacent carbon-marked needle biopsy track (arrowhead). India ink was painted at the margins (thin arrows) prior to excision of the specimen and should be distinguished from the carbon-marked track.

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Representative photomicrograph of the histologic findings in the same specimen as in Figure 2. Infiltrating tubular carcinoma (arrow) is shown, with adjacent black carbon deposition in the scar reaction of the needle biopsy track. Note that the carbon-marked track (arrowheads) leads directly to the tumor and does not interfere with the interpretation of the histologic findings.

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## LIQUID MARKER FOR PREOPERATIVE BREAST LOCALIZATION

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 *The importance of accurate preoperative localization of mammographically detected lesions has been proved.*

- ▶ *There are several mammographically guided methods, all with their advocates. A survey of localization methods that are currently in use revealed that mostly a needle and wire system is used, while a dye alone, commonly methylene blue, is used in 8% of cases.*
- ▶ *Wire dislodgement and widespread diffusion of the liquid marker into surrounding tissue are well known disadvantages connected with the above methods*
- ▶ *The need for a stable marker has been pursued through a special biocompatible charcoal powder suspension specifically treated for this purpose.*
- ▶ *CARBO-REP® is available in pre-filled 2 ml vials, easy to inject into the chosen site to mark the path to the lesion for the surgeon.*
- ▶ *One more interesting advocate of CARBO-REP® is the possibility to indicate the exact location of the site of initial biopsy in instance of the subsequent development of a mammographically occult cancer in proximity to the initial biopsy site.*

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