

# INSTRUCTION FOR USE OF SKIN BIOPSY DEVICE DERMO-PUNCH®



## WARNING:

- Store in a dry clean place, away from heat sources.
- Before use ensure that the packaging is intact: if perforated, damaged or open discard the instrument
- Check the expiry date and that the model corresponds
- Peel open the package
- Pay attention to not compromise the sterility of the instrument before use.
- In the event of a product defect or malfunction, complete the yellow claim form to be found in the product and conserve the defective instrument.

## N.B.:

The use of this instrument is reserved to qualified medical personnel who are aware of its application  
The instrument is sterile (sterilized by ETO GAS) single use and must be discarded after use (Circ.Min:Sanità 403/9.5/1146-L. 10.81)  
**Sterylab declines any and all responsibility in the event of improper use or reutilisation.**

## PROCEDURE

- 1) Remove the protection from the instrument
- 2) Make a local anesthesia where the skin must be cut
- 3) Make an incision with DERMOPUNCH, where the skin must be drawn, by doing a rotatory motion in a clockwise direction and counter-clockwise direction; meanwhile apply a slight pressure on the skin to permit the penetration of the instrument in the tissue.
- 4) After having reached the desired depth, pull out the DERMOPUNCH.
- 5) Cut the strip of skin, previously incised, with pincers and scalpel (not provided), to draw it.
- 6) Suture the lips of injury, if necessary.