





INSTRUCTIONS FOR USE OF THE NEEDLE GUIDE BUSHES

CE 0426

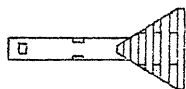

EA for ultrasound, to be used with the ESAOTE-ANSALDO (HITACHI) transducers

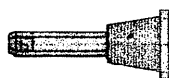

CGR for stereotaxis, to be used with CGR mammographs

- WARNING:**
- Store in a dry clean place away from heat sources
 - Before use ensure that the packaging is intact: if perforated, damaged or open discard the instrument
 - Check the expiry date and that the model corresponds
 - Peel open the package
 - Pay attention to not compromise the sterility of the instrument before use
 - In the event of a product defect or malfunction complete the yellow claim form to be found in the product carton and conserve the defective instrument

N.B.:

The use of this instrument is reserved to qualified medical personnel who are aware of its application.

The instrument is sterile (sterilized by ETO GAS) single use and should be discarded after use (Circ.Min:Sanità 403/9.5/1146-L. 10.81)

Sterylab declines any and all responsibility in the event of improper use or reutilisation.

Before proceeding to use the needle guide bushes verify their correct combination with the ultrasound transducer (Mod. EA) or the mammograph.

The colour of the guide bush is according to the international code and indicates the calibre of needle to be used.

GAUGE	\varnothing mm	colour
14	2,0	dark green
16	1,6	white
18	1,2	pink
20	0,9	yellow
21	0,8	light green
22	0,7	black