




INSTRUCTION FOR USE OF THE GALACTOGRAPHY SET SGL

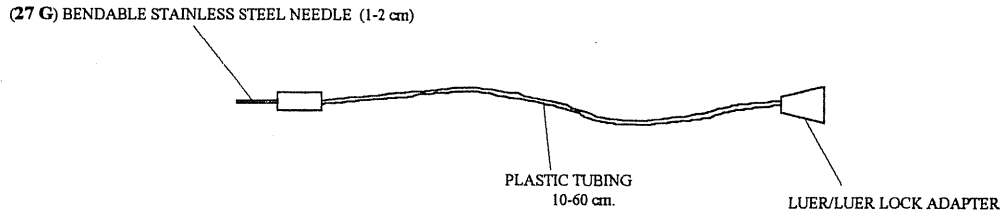

- WARNING:**
- Store in a dry clean place away from heat sources
 - Before use ensure that the packaging is intact: if perforated, damaged or open discard the instrument
 - Check the expiry date and that the model corresponds
 - Peel open the package
 - Pay attention to not compromise the sterility of the instrument before use
 - In the event of a product defect or malfunction complete the yellow claim form to be found in the product carton and conserve the defective instrument

N.B.:

The use of this instrument is reserved to qualified medical personnel who are aware of its application.

The instrument is sterile (sterilized by ETO GAS) single use and should be discarded after use (Circ.Min:Sanità 403/9.5/1146-L. 10.81)

Sterylab declines any and all responsibility in the event of improper use or reutilisation.

The SGL set is available in two models:

- 1 cm or 2 cm needle length
- 10 cm or 60 cm plastic tubing length

If necessary the needle can be moderately bent before use.

After connecting the syringe (luer/luer lock adapter) containing the contrast medium, flush the set with contrast in order to remove all air before initiating the procedure.