



INSTRUCTIONS FOR USE OF THE DISPOSABLE ISTEROMETRE IS-1



- WARNING:**
- Store in a dry clean place away from heat sources
 - Before use ensure that the packaging is intact: if perforated, damaged or open discard the instrument
 - Check the expiry date and that the model corresponds
 - Peel open the package
 - Pay attention to not compromise the sterility of the instrument before use
 - In the event of a product defect or malfunction complete the yellow claim form to be found in the product carton and conserve the defective instrument

N.B.:

The use of this instrument is reserved to qualified medical personnel who are aware of its application.
The instrument is sterile (sterilized by ETO GAS) single use and should be discarded after use (Circ.Min:Sanità 403/9.5/1146-L. 10.81)
Sterylab declines any and all responsibility in the event of improper use or reutilisation.

- 1) Introduce into the cervical canal the extremity A) up until it's arrest against the cervix
- 2) Push forward the measuring rod. A) using the handle B) until it reaches the end of the uterine cavity
- 3) Read from the centimeter scale C). This indicator indicates the depth from the cervix to the bottom of the endometrium *

* The degree of precision of the scale is $\pm 3\text{mm}$.