




INSTRUCTIONS FOR USE OF THE IVG DISPOSABLE CURETTE KARMAN

CE 0426

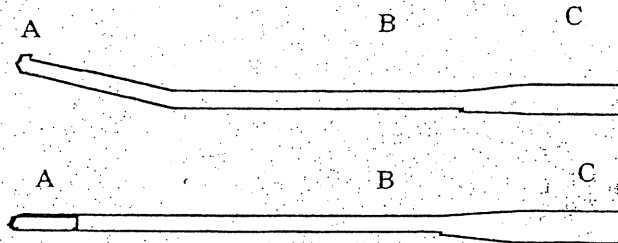

- WARNING:**
- Store in a dry clean place away from heat sources
 - Before use ensure that the packaging is intact: if perforated, damaged or open discard the instrument
 - Check the expiry date and that the model corresponds
 - Peel open the package
 - Pay attention to not compromise the sterility of the instrument before use
 - In the event of a product defect or malfunction complete the yellow claim form to be found in the product carton and conserve the defective instrument

N.B.:

The use of this instrument is reserved to qualified medical personnel who are aware of its application.

The instrument is sterile (sterilized by ETO GAS) single use and should be discarded after use (Circ.Min.Sanita 403/9.5/1146-L. 10.81)

Sterylab declines any and all responsibility in the event of improper use or reutilisation.

N.B.: Angled, straight, rigid, flexible curettes are all available; ensure that the model is the right one and that the calibre (number next to the opaque tract B) correspond.

- 1) Before the intervention measure the uterine depth using an hysterometer (e.g. our disposable IS-1)
- 2) Connect the standard conical connector C) to the suction device
- 3) Introduce the cannula into the endometrial cavity utilising the opaque tract B) of the cannula (from 10 to 12 cm. from the tip) and turn on the aspirator
- 4) Adhere the mouth A) of the cannula to the endometrium, rotate the curette and aspirate totally the foetus